



English

Instruction and Service Manual

Models 920 and 920M
Hand Held Pulse Oximeters



About the Manual

Boxes draw attention to contraindications, warnings, and cautions.

The boxed messages are important. Be sure to read them.

The information in this manual has been carefully checked and is believed to be accurate. In the interest of continued product development, RESPIRONICS, INC. reserves the right to make changes and improvements in this manual and the products it describes at any time, without notice or obligation.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Respironics Georgia

175 Chastain Meadows Court
Kennesaw, Georgia 30144-3724
USA

Customer Service Telephone: 1-800-345-6443 (USA and Canada)

Customer Service Fax: 1-800-886-0245 (USA and Canada)

www.respironics.com

Respironics Deutschland

Gewerbestrasse 17
82211 Herrsching
Germany
Telephone: 49 815 293 060
Fax: 49 815 293 0618

Respironics Asia Pacific

3/F, Microtron Building
38 Hung To Road
Kwun Tong, Kowloon, Hong Kong
Telephone: 852-2771-1886
Fax: 852-2770-7557


References to "920" in this manual shall imply Models 920 and 920M.
References to "Respironics" in this manual shall imply Respironics, Inc.

RESPIRONICS is a registered trademark of Respironics, Inc.

Copyright © 1999 Respironics, Inc. All rights reserved.

English

Table of Contents

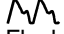I. Contraindications, Warnings, and Cautions	1
II. Unpacking Your Pulse Oximeter	3
III. Introduction	4
A. Indications for Use	4
B. General	4
IV. Operating Instructions	5
A. Batteries	5
B. General	7
1. Connecting the Sensors	7
2. Turning on the Pulse Oximeter	7
3. Verifying Operation	9
4. Cleaning the Pulse Oximeter	9
V. Features	10
A. Controls	10
1. Power	10
2. Display Brightness	10
3. Setup Mode	11
a. Printer Settings	11
b. Calendar Settings	13
c. Clock Settings	14
B. Displays	15
1. SpO ₂ Display	15
2. ♥ (Pulse Rate) Display	15
3.  (Perfusion) Indicator	15
4. Flashing Display	16
5. Dash in the SpO ₂ Display	16
C. Printer/Serial Output	17
D. Memory Option (920M only)	18
1. Recording Sessions	18
2. Memory Output Mode	19
VI. Pulse Oximeter Sensors	21
A. General Sensor Description	21
B. Finger Clip Sensors	24
1. Adult Articulated Finger Clip Sensor	24
2. Pediatric Finger Clip Sensor	24
C. Flex Sensors	25
1. Adult Flex Sensor	25
2. Infant Flex Sensor	26
3. Neonatal Flex Sensor	26
D. Ear Clip and Reflectance Sensors	27
1. Ear Clip Sensor	27
2. Reflectance Sensor	28
E. Adult and Pediatric Disposable Sensors	28

F. Infant and Neonatal Disposable Sensors	30
1. Infant Flexi-Form Sensor.....	30
2. Neonatal Flexi-Form Sensor	31
G. Cleaning the Sensors	33
H. Sensor Compatibility	33
VII. Theory of Operation	34
VIII. Specifications	36
IX. Service	38
X. Warranty	39
XI. Accessories.....	40
XII. Troubleshooting Guide	42





Tables

Table I: Printer, clock, and calendar mode parameters.....	11
Table II: 920P Printer modes.....	12
Table III: Printer/Sensor interface assignments.....	17
Table IV: 920M memory data format.....	20
Table V: Reusable sensor reference.....	22
Table VI: Disposable sensor reference.....	23
Table VII: Sensor accessory reference.....	23

Figures

Figure I: Replacing batteries in the 920.....	6
Figure II: Connecting sensors to the 920.....	7
Figure III: Front view of the 920.....	8
Figure IV: Rear view of the 920.....	8
Figure V: Flowchart for setting the printer mode. ..	12
Figure VI: Flowchart for setting the calendar.....	13
Figure VII: Flowchart for setting the clock.....	14
Figure VIII: Adult Articulated Finger Clip Sensor....	24
Figure IX: Pediatric Finger Clip Sensor.....	24
Figure X: Adult Flex Sensor.....	25
Figure XI: Flex Sensor with tape strain relief.....	25
Figure XII: Infant Flex Sensor on big toe.....	26
Figure XIII: Neonatal Flex Sensor on foot.....	27
Figure XIV: Ear Clip Sensor.....	27
Figure XV: Finger Flexi-Form Sensor.....	29
Figure XVI: Infant Flexi-Form Sensor.....	31
Figure XVII: Neonatal Flexi-Form Sensor.....	32

Glossary of Symbols

Symbol	Definition of Symbol
	ATTENTION Consult ACCOMPANYING DOCUMENTS
	Type BF Applied Part: (Patient isolation from electrical shock).
	Canadian Standards Association
UL	Underwriters Laboratories
	CE Marking indicating conformance to EC directive No. 93/42/EEC concerning medical devices.

English

I. Contraindications, Warnings, and Cautions



CONTRAINDICATIONS

- Do not operate in an explosive atmosphere.
- Do not operate the RESPIRONICS 920 in an MRI environment.
- Do not use the 920 in a situation where alarms are required. The 920 and 920M models have no audible alarms.



WARNING

- The 920 is intended only as an adjunct in patient assessment. It must be used in conjunction with clinical signs and symptoms.
- Use only Respironics-manufactured sensors. These sensors are manufactured to meet the calibration requirements for Respironics pulse oximeters. Use of other manufacturer's sensors may cause improper pulse oximeter performance.
- Check application site **frequently** to determine circulation, positioning, and skin sensitivity of the patient. Each patient's sensitivity to Respironics sensors may vary depending on their medical status or the condition of their skin.
- Use of Respironics double-backed adhesive strips or the Hydrogel tape strips should be discontinued if the patient exhibits allergic reactions to the adhesive material.
- Do not stretch the adhesive tape while applying the sensors. This may cause inaccurate readings or skin blisters.

 **CAUTION**

- Federal law (USA) restricts this device to sale by or on the order of a physician.
- Read this manual carefully before use of the 920.
- Carefully read the instructional insert provided with the sensor before use.
- The 920 is intended for spot-checking or continuous monitoring by an attending health care professional. Because the 920 has no audible alarms, international labeling requirements dictate it be labeled "Not for continuous monitoring."
- The 920 must be able to measure the pulse properly to obtain accurate SpO₂ measurement. Verify that nothing is hindering the pulse measurement before relying on the SpO₂ measurement.
- Fingernail polish may reduce light transmission and thereby affect SpO₂ accuracy.
- The 920 may not work on all patients. If you are unable to achieve stable readings, discontinue use.
- The 920 is sensitive and must be repaired by knowledgeable and specially trained personnel only.
- The 920 may interpret motion artifact of sufficient amplitude and regularity as good perfusion (green).
- The 920 is calibrated to determine the percentage of arterial oxygen saturation of functional hemoglobin. Significant levels of dysfunctional hemoglobin such as carboxyhemoglobin or methemoglobin may affect the accuracy of the measurement.
- Cardiogreen and other intravascular dyes, depending on the concentration, may affect the accuracy of the SpO₂ measurement.
- Do not use different types of batteries at the same time. Do not mix fully charged and partially charged cells at the same time. These actions may cause the batteries to leak.
- Do not immerse the 920 or Respironics sensors in liquid to clean.
- Do not use caustic or abrasive cleaning agents.
- Ear Clip and Reflectance sensors are not recommended for pediatric or neonatal use. The accuracy of the sensors has not been established for pediatric or neonatal use.
- Do not remove any covers other than the battery cover when battery replacement is necessary. There are no user serviceable parts inside other than the replaceable batteries.
- Alkaline batteries may leak or explode if used or disposed of improperly.

II. Unpacking Your Pulse Oximeter

Contact the carrier immediately if the shipping carton for the 920 is damaged. Carefully unpack the instrument and its accessories. Confirm that the items listed below are packed with the 920 Hand Held Pulse Oximeter. The 920/920M shipment includes:

- Model 920 or 920M Hand Held Pulse Oximeter
- Instruction and Service Manual for Models 920 and 920M
- Six AA batteries
- Model 936 Sensor (Adult Articulated Finger Clip)

If any item on this list is missing or damaged, do not use the pulse oximeter. Contact your local distributor or, if you do not know your local distributor, contact Respirationics.

English

III. Introduction

A. Indications for Use

The 920 Hand Held Pulse Oximeter is intended to be used for monitoring oxygen saturation and pulse rate for adult, pediatric, and neonatal patients in hospital, ambulatory, home, and EMS environments. The 920 may be used for spot checking and/or continuous monitoring when attended by a healthcare professional. The variety of individual sensors available must be checked frequently to ensure proper circulation and application.

B. General

The 920 Hand Held Pulse Oximeter is small and lightweight. The 920 has visual alarms for tracking patient status. It typically will operate for 100 hours continuously between battery replacements. **The 920 requires no routine calibration or maintenance.**

The 920 determines arterial oxyhemoglobin saturation (%SpO₂) by measuring the absorption of red and infrared light passed through tissue. Changes in absorption caused by pulsation of blood in the vascular bed are used to determine arterial saturation and pulse rate.

Oxygen saturation and pulse rate are displayed on light emitting diode (LED) digital displays. On each detected pulse, the perfusion LED flashes. Patient perfusion signals are graded as good, marginal or inadequate and are indicated as such by the LED flashing green, yellow, or red. This simple method gives the user a pulse-by-pulse visual indication of waveform signal quality without requiring the user to perform complex waveform analysis during critical patient care situations.


Sensor disconnect or malfunction is indicated by lack of good perfusion flashes and/or a dash to the left of the SpO₂ value on the LED display. Ultimately, if adequate perfusion pulses are not received, the SpO₂ and Pulse Rate (©) numerical values will be replaced by dashes. When the batteries are low, the digital displays will blink.

The 920 Pulse Oximeter may be used with all Respironics pulse oximeter sensors (except fiber optic sensors) enabling proper operation for nearly every patient.

NOTE: *Respironics sensors do not contain natural rubber latex.*

IV. Operating Instructions

A. Batteries

 CAUTION	<ul style="list-style-type: none">• Do not remove any covers other than the battery cover when battery replacement is necessary. There are no user serviceable parts inside other than the replaceable batteries.• Alkaline batteries may leak or explode if used or disposed of improperly.• Do not use different types of batteries at the same time. In addition, do not mix fully charged and partially charged cells at the same time. These actions may cause the batteries to leak.
--	--

English

The 920 Pulse Oximeter is powered by 6 AA Alkaline cells that will typically provide 100 hours of continuous operation. **The 920 indicates when the batteries are low by flashing the digital displays once each second.** When the displays begin flashing, the batteries should be replaced as soon as possible. Replace the batteries by removing the battery door on the bottom of the 920. Be sure to follow the polarity markings on the rear label of the pulse oximeter when installing new batteries. Refer to Figure I for an illustration of battery replacement.

Rechargeable Nickel Cadmium batteries may be used in the 920 if desired. Since NiCad batteries have less than half the capacity of alkaline batteries, the batteries will have to be recharged more often than every 100 hours.

NOTE: *Reducing the LED display brightness can extend battery life up to 100%! Refer to Section V (A. 2.).*

NOTE: *Replacing batteries erases the clock settings of the 920. The memory of the 920M will also be erased when the batteries are replaced. These settings should be reset after the batteries are replaced. To do this, refer to Section V (A. 3.).*

NOTE: *Batteries should be removed if the 920 is going to be stored for more than 30 days. Batteries may leak if left in the device for a long period of time.*

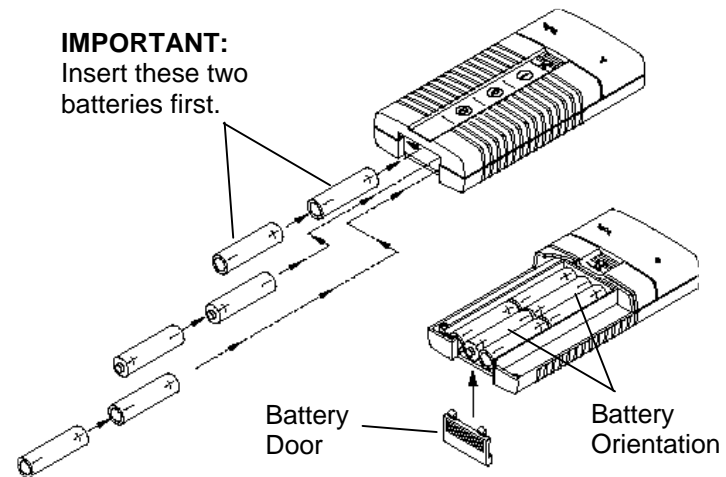


Figure I: Replacing batteries in the 920.

B. General

The 920 Hand Held Pulse Oximeter is portable and is intended for attended patient monitoring by trained personnel. It displays numerical values for oxygen saturation and pulse rate. Because the 920 Pulse Oximeter has no patient alarms, the user must frequently observe the SpO₂ and Pulse Rate displays.

1. Connecting the Sensors

Connect the sensor to its 9-pin mating jack on the top of the 920 as shown in Figure II. To select a sensor type, refer to Section VI (A). If additional cable length is necessary, connect the Model 927 Patient Cable between the sensor and the 920 Pulse Oximeter. Position the appropriate sensor on the patient. Refer to Section VI for sensor positioning information.

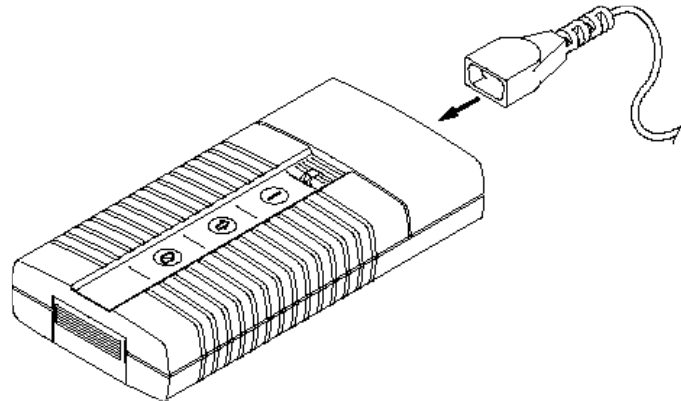


Figure II: Connecting sensors to the 920.


2. Turning on the Pulse Oximeter

Turn on the 920 Hand Held Pulse Oximeter by depressing the “ \odot ” button on the front of the pulse oximeter. Refer to Figure III.


When the 920 is powered on, the SpO₂ and \odot displays will cycle through the following sequence before displaying valid data values:

- “**888 888**”
- time saved in memory in hours and minutes
- software revision number
- “- -”


3. Verifying Operation

 CAUTION	<ul style="list-style-type: none">• The 920 must be able to measure the pulse properly to obtain accurate SpO₂ measurement. Verify that nothing is hindering the pulse measurement before relying on the SpO₂ measurement.
--	--

English

Verify that the sensor is properly positioned. Ensure the system is sensing adequate perfusion by observing that the  (perfusion) indicator is blinking green, and the blinking is correlated to the pulse rate for 10 seconds. Should the perfusion light be red or yellow or flashing erratically, reposition the sensor or try a different sensor.

4. Cleaning the Pulse Oximeter

 CAUTION	<ul style="list-style-type: none">• Do not immerse the 920 in liquid to clean.• Do not use caustic or abrasive cleaning agents.
--	--

The 920 Pulse Oximeter may be cleaned with a mild detergent and a soft cloth or with an isopropyl alcohol wipe. Allow enough time for the 920 to dry thoroughly before reusing.

V. Features

A. Controls

All functions of the 920 are controlled by switches found on the front of the unit. Refer to Figure III for an illustration of these switches.

1. Power

Pressing the ON switch (“ \odot ”) causes power to be applied to all internal circuitry. Pressing the OFF switch (“ \AE ”) causes power to be removed from the displays and puts the pulse oximetry circuitry into a low power standby mode.

In order to conserve battery life, the 920 will automatically power off after 10 minutes of inactivity. Inactivity is indicated by dashes on the displays and is caused by:

- no sensor connected to the pulse oximeter
- patient pulse too low
- sensor not attached to a patient

Each time a reading is displayed, the 10-minute timer is restarted.

The “ \odot ” switch has additional clock and printer mode setting functions when used in conjunction with the “ \uparrow ” switch. Refer to Section 3 of this section for Setup Mode instructions.

2. Display Brightness

The arrow switch (“ \uparrow ”) causes the brightness of the digital displays to change. When powered up, the digital display defaults to the maximum brightness. Pressing the “ \uparrow ” will advance the brightness to the lowest setting, and each subsequent press will advance the brightness through 8 different settings. The function is circular which means it will cycle through the entire brightness range and then start at the beginning again. The lower light brightness may be used to preserve battery life. The higher light intensity may be used to view the displays from a distance.

NOTE: *Reducing the LED display brightness can extend battery life up to 100%!*

The “ \uparrow ” switch has additional clock mode setting functions when used in conjunction with the “ \odot ” switch. Refer to Section 3 of this section for Setup Mode instructions.

3. Setup Mode

Setup mode is used to control the internal time-of-day clock and the external real-time printer (purchased separately). The setup mode is initiated by holding the “ $\hat{\cup}$ ” switch when the unit is turned on by pressing the “ $\hat{\circ}$ ” switch. In setup mode, the “ $\hat{\circ}$ ” switch and the “ $\hat{\cup}$ ” switch are used to make the selections.

Advance to the next sequential mode by pressing the “ $\hat{\circ}$ ” switch. Each time the “ $\hat{\cup}$ ” switch is pressed, the number on the ♥ display will increment. It starts with the current value stored in memory for the parameter designated in the SpO₂ display. When the correct value appears in the ♥ display, pressing the “ $\hat{\circ}$ ” switch will advance the SpO₂ display to the next sequential parameter as listed in Table I. This process is continued until all parameters are set. The settings can be easily checked, since the first value displayed for each parameter represents the current setting. When the setting sequence has been completed, the 920 exits the setup mode and begins normal operation.

Sequence	Appears in SpO ₂ Display	Range of Values	
		From:	To:
Printer	“Prn”	00	15
Year	“y”	00	99
Month	“nn”	00	12
Day	“d”	01	31
Hours	“h”	00	23
Minutes	“nn”	00	59

Table I: Printer, clock, and calendar mode parameters.

a. Printer Settings

NOTE: *The 920P prints out real-time data only. Data stored in the 920M can not be downloaded to the 920P.*

“Prn” will appear in the SpO₂ display indicating print setup mode. There are 16 options for the printer mode: 00 through 15. Each printer mode is explained in Table II. The modes determine how often data is written out to the printer and the format of data written to the printer. Refer to Figure V for a flowchart of setting the printer mode. When the printer setting sequence has been completed, the 920 continues to the calendar settings (refer to section b of this section).

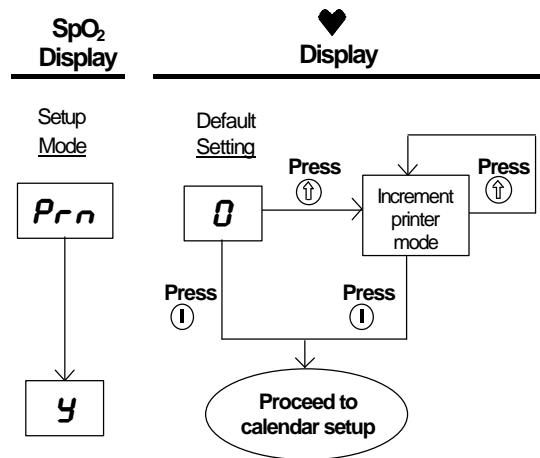


Figure V: Flowchart for setting the printer mode.

Printer Mode	Seconds per data point	Minimum SpO ₂ printed?*	Touch print capability?*
00	10	No	Yes
01	30	No	Yes
02	120	No	Yes
03	10	Yes	Yes
04	30	Yes	Yes
05	120	Yes	Yes
15	---	No	Yes

* For modes where the minimum SpO₂ data is written, there are two lines of data written for each data output. The first line contains the minimum value for SpO₂ since the last printout, and the second line contains the current data values.

** The touch print mode enables the user to print out data at any time. This is activated by pressing the “o” switch.

Table II: 920P Printer modes.

NOTE: Printer modes 06 – 14 are not used at this time. They are reserved for future development.

b. Calendar Settings

After the printer setting has been determined in the setup mode, “y” will appear in the SpO₂ display indicating calendar setup mode for the year. The year may be set to “00” through “99”. After selecting the year, the display will show “nn” indicating the setup mode for the month. The month may be set to “00” through “12”. After selecting the month, the display will show “d” indicating the setup mode for the day of the month. The day may be set to “01” through “31”. Refer to Figure VI for a flowchart of setting the calendar. When the calendar setting sequence has been completed, the 920 continues to the clock settings (refer to section c. of this section).

NOTE: Setting the month to “00” disables the clock function and helps conserve battery life.

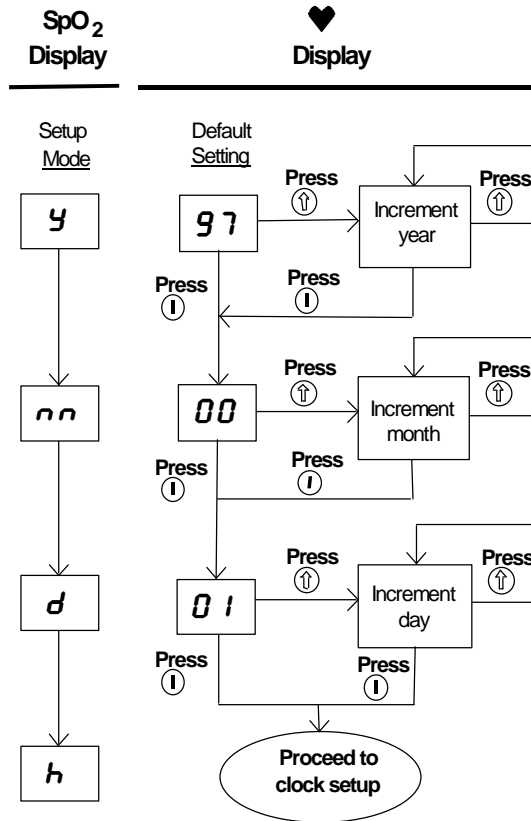


Figure VI: Flowchart for setting the calendar.

c. Clock Settings

After the calendar settings have been determined in the setup mode, “h” will appear in the SpO₂ display indicating clock setup mode for the hour. The hour may be set to “00” through “23”. After selecting the hour, the display will show “nn” indicating the setup mode for the minutes. The minutes may be set to “00” through “59”. After selecting the minutes, the display will return to normal operation. Refer to Figure VII for a flowchart of setting the clock.

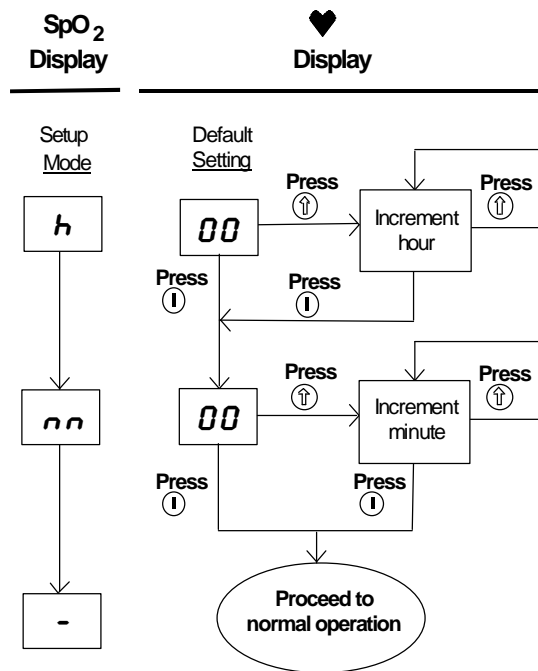


Figure VII: Flowchart for setting the clock.

The printer modes are explained in Table II. The modes determine how often data is written out to the printer and the format of data written to the printer. For modes where the minimum SpO₂ data is written, there are two lines of data written for each data output. The first line contains the minimum value for SpO₂ since the last printout, and the second line contains the current data values. The touch print mode enables the user to print out data at any time. This is activated by pressing the “ô” switch.

B. Displays


1. SpO₂ Display

The left digital display is a 3-digit light emitting diode (LED) digital display that indicates oxygen saturation percentage. Refer to Figure III. Oxygen saturation is calculated by the methods outlined in Section VII.


2. © (Pulse Rate) Display

The right digital display is a 3-digit LED digital display that indicates pulse rate in pulses per minute. Refer to Figure III.

3. (Perfusion) Indicator

The perfusion indicator (identified by the waveform symbol ) will flash once for each pulse while measuring oxygen saturation. The perfusion indicator changes color to indicate changes in the pulse waveform signal that may affect the SpO₂ data.

The perfusion indicator may blink one of three colors: green, yellow, or red. These colors are similar to the colors of a stoplight such that:

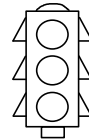

Red indicates the pulse amplitude is too small. During red perfusion, SpO₂ and pulse rate values are not updated. After ten seconds, the values are replaced with dashes indicating SpO₂ measurement is not possible.

Yellow indicates the pulse waveform amplitude is marginal or the pulse oximeter has detected artifact. Although SpO₂ data is acceptable, corrective measures should be considered to improve sensor placement, change sensor type, or reduce patient movement.

Green indicates the pulse waveform signal is of good quality and SpO₂ data is accurate.



CAUTION

- The 920 may interpret motion artifact of sufficient amplitude and regularity as good perfusion (green).

4. Flashing Display

The numerical displays will flash once each second if the 920 determines that a battery low condition exists. Replace all six batteries immediately.

NOTE: *Inaccurate SpO₂ and/or pulse rate measurement may result if the 920 is operated in a low battery condition.*

5. Dash in the SpO₂ Display

If the 920 determines that a sensor fault exists (sensor disconnect, sensor failure, or sensor dislodgment), a dash (-) appears in the left-most digit of the SpO₂ display. The readings that are being displayed will remain unchanged while the sensor fault exists. If the sensor fault is not corrected, dashes will be displayed 10 seconds after the minus sign appears.

C. Printer/Serial Output

Both the 920 and 920M Hand Held Pulse Oximeters provide output capability to a custom printer via the 9-pin Sub-D connector. This connector serves as a sensor-input connector as well as a printer interconnect device. The 9-pin Sub-D connector pin assignments are listed in Table III.

<u>Pin Number</u>	<u>Assignment</u>
1	Battery Voltage
2	Infrared Anode, Red Cathode
3	Infrared Cathode, Red Anode
4	Serial Data, TTL Levels
5	Detector Anode
6	Logic Level
7	Cable Shield
8	Coaxial Shield
9	Detector Cathode, +5 V

Table III: Printer/Sensor interface assignments.

The information from the 920 in the real-time mode is sent in an ASCII serial format at 9600 baud with 9 data bits, 1 start bit, and 1 stop bit, odd parity. The data is output at a rate of once per second.

NOTE: *The 9th data bit is used for parity in memory playback mode. In real-time mode, it is always set to the mark condition. Therefore the real-time data may be read as 8 data bits, no parity.*

The data printed by the 920P printer is in the following format:

HH:MM:SS SPO2=XXX HR=YYY

where “HH” represents the hour the real-time clock is set to, “MM” represents the minutes, “SS” represents the seconds, “XXX” represents the SpO₂ value, and “YYY” represents the heart rate. The SpO₂ and heart rate will be displayed as “---” if there is no data available for the data reading.

D. Memory Option (920M only)

The memory option is identified by the "M" in the model number (i.e., 920M as opposed to the 920). This model number is located just above the serial number on the back of the unit. The 920M Pulse Oximeter can collect and store up to eighteen hours of SpO₂ and pulse rate information.

The solid-state memory in the 920M functions much like the "endless loop" tapes that are used in some telephone answering machines. When the memory fills up, the unit begins overwriting the oldest locations with the latest data.

Each time the 920M is powered up, the current time/date information (if the clock is set properly) is stored in memory to allow quick differentiation of recording sessions. Patient SpO₂ and pulse rate are sampled and stored every four seconds. The stored resolution of the oxygen saturation is in 1% increments in the range of 0 to 100 %. The stored pulse rate ranges from 18 to 300 BPM. The stored values have a resolution of 1 BPM from 18 to 200 and a resolution of 2 BPM from 201 to 300.

During the printing of the data, the last data recorded is the first data printed. For example, the last four minutes of data recorded would be the first four minutes of printout.

1. Recording Sessions

Each time the 920M is turned on (except while setting the clock) data is automatically collected.

NOTE: *Only recording sessions greater than one minute in length are kept in memory for later printing.*

2. Memory Output Mode

To output the data stored in the memory of the 920M, start with the unit OFF and then:

- 1) Hold the “↑” switch while pressing the “ô” switch;
- 2) Release the “↑” and “ô” switches when “**888 888**” is displayed in the SpO₂ and pulse rate LEDs;
- 3) Observe “**P r n xx**” will be displayed in the SpO₂ and ♥ LEDs;
- 4) Data is automatically transferred from the memory in approximately 8 to 10 minutes.

Data is transferred at a rate of 20 minutes of collected data per second. An 18 hour recording session (the maximum memory saved) is transferred in approximately 1 minute. After all the data is transferred, the 920M should be shut off prior to collecting new patient data. The patient information is held in memory as long as the batteries are good, so if the memory has to be cleared, remove the battery door for a period of 60 seconds or longer. Outputting the memory does not clear any data from the memory.

NOTE: *The memory is cleared each time the batteries are changed.*

The format of the data transferred is given in Table IV. The size of this file will depend on the amount of data saved in the memory. The most recent data is transferred first. The memory data format is in binary. Bad data is represented by FF (hexadecimal) or 255 (decimal). If the memory “wrapped around” (the recording time exceeded 18 hours) and the final (i.e., the oldest) file of data has been truncated, the final start time will be represented by zeroes and the start times for that file will then not match up.

920M Memory Data Format


DATA 1	DATA 2	CHECK SUM**	NOTE
\$FE	\$FE	\$FC	Header of first file
month	Day	XX	Current time
year	Minute	XX	
second	Hour	XX	
month	Day	XX	Stop time of first file saved in memory
year	Minute	XX	
second	Hour	XX	
month	Day	XX	Start time of first file saved in memory
year	Minute	XX	
second	Hour	XX	
pulse rate	SpO ₂	XX	Data in first file
pulse rate	SpO ₂	XX	
pulse rate	SpO ₂	XX	
•	•	•	
•	•	•	
•	•	•	
pulse rate	SpO ₂	XX	
\$FE	\$FE	\$FC	Header of second file
month	Day	XX	Start time of first file saved in memory
year	Minute	XX	
second	Hour	XX	
month	Day	XX	Stop time of file <i>N</i> saved in memory
year	Minute	XX	
second	Hour	XX	
month	Day	XX	Start time of file <i>N</i> saved in memory
year	Minute	XX	
second	Hour	XX	
pulse rate	SpO ₂	XX	Data in file <i>N</i>
pulse rate	SpO ₂	XX	
pulse rate	SpO ₂	XX	
•	•	•	
•	•	•	
•	•	•	
pulse rate	SpO ₂	XX	
\$FE	\$FE	\$FC	Header of file <i>N+1</i>
month	Day	XX	Start time of file <i>N</i> saved in memory
year	Minute	XX	
second	Hour	XX	
0	0	0	End of data marker (18 zeroes)
0	0	0	
0	0	0	
0	0	0	
0	0	0	
0	0	0	

NOTE: ** CHECKSUM = DATA1 + DATA2


The data represented by the shaded area is repeated for additional data blocks that are saved in the 920M's memory. *N* represents the "next file" saved in the memory.

Table IV: 920M memory data format.

VI. Pulse Oximeter Sensors

 WARNING	<ul style="list-style-type: none">• Use only Respirationics-manufactured sensors. These sensors are manufactured to meet the calibration requirements for Respirationics pulse oximeters. Use of other manufacturer's sensors may cause improper pulse oximeter performance.• Use of Respirationics double-backed adhesive strips or the Hydrogel tape strips should be discontinued if the patient exhibits allergic reactions to the adhesive material.• Check application site frequently to determine circulation, positioning, and skin sensitivity of the patient. Each patient's sensitivity to Respirationics sensors may vary depending on their medical status or the condition of their skin.
--	---

English

 CAUTION	<ul style="list-style-type: none">• Before use, carefully read the instructional insert provided with the sensor.
--	---

NOTE: *Respirationics sensors do **not** contain natural rubber latex.*

A. General Sensor Description

The 920 features sensors which are ideal for nearly every application. Refer to Tables V and VI for a complete list of Respirationics sensors and specific information on each sensor. Each sensor is designed for a specific site application and specific patient size and weight range. Reusable sensors are designed for multiple patient use, while disposable sensors are designed for single patient use.

Factors that may degrade performance:

- excessive ambient light
- excessive motion
- electrosurgical interference
- arterial catheters, blood pressure cuffs, infusion lines, etc.
- moisture in the sensor
- improperly attached sensor
- incorrect sensor type
- poor perfusion
- venous pulsations
- anemia or low hemoglobin concentrations
- cardiovascular dyes
- sensor not at heart level

Respironics Reusable Sensor Reference

Sensor Description	Respironics Model Number	Approximate Patient Weight Range	Preferred Site	Application
Adult Articulated Finger Clip	936	greater than 30 kg (greater than 66 lb.)	Index, middle, or ring finger	<ul style="list-style-type: none"> • spot checking • short-term continuous use • for adults or adolescents
Pediatric Finger Clip	936P	greater than 8 kg (greater than 18 lb.)	Index, middle, or ring finger	<ul style="list-style-type: none"> • spot checking • short-term continuous use • for children or small adults
Ear Clip ¹	948	greater than 40 kg (greater than 88 lb.)	Ear lobe	<ul style="list-style-type: none"> • spot checking • short-term continuous use • stress testing
Reflectance ¹	951	greater than 30 kg (greater than 66 lb.)	Middle of forehead	<ul style="list-style-type: none"> • continuous monitoring • stress testing
Adult Flex	953	greater than 20 kg (greater than 44 lb.)	Index, middle, or ring finger	<ul style="list-style-type: none"> • extended duration • motion situations • for adults
Infant Flex	934	2 to 20 kg (4 to 44 lb.)	Great toe	<ul style="list-style-type: none"> • extended duration • motion situations • for infants
Neonatal Flex	935	less than 2 kg (less than 4 lb.)	Lateral aspect of foot	<ul style="list-style-type: none"> • extended duration • motion situations • for neonates

Table V: Reusable sensor reference.

¹ **Reflectance** and **Ear Clip** sensors generally do not perform as well as sensors located on the fingers or toes. They are not recommended for applications where the best possible SpO₂ accuracy is important.

NOTE: *The reusable sensors must be secured to the patient using either Velcro®-type straps or medical tapes. Respironics **Tape Strips** or **Hydrogel Tape Strips** are recommended to stabilize the reusable sensors.*

Respironics Disposable Sensor Reference

Sensor Description	Respironics Model Number	Patient Weight Range	Preferred Application	Application
Adult Flexi-Form®	953D	greater than 30 kg (greater than 66 lb.)	Index, middle, or ring finger	<ul style="list-style-type: none"> extended monitoring self-adhesive motion situations for adults
Pediatric Flexi-Form®	937D	10 to 40 kg (22 to 88 lb.)	Index, middle, or ring finger	<ul style="list-style-type: none"> extended duration self-adhesive motion situations for children
Infant Flexi-Form®	934D	2 to 20 kg (4 to 44 lb.)	Great toe	<ul style="list-style-type: none"> extended duration self-adhesive motion situations for infants
Neonatal Flexi-Form®	935D	less than 2 kg (less than 4 lb.)	Lateral aspect of foot	<ul style="list-style-type: none"> extended duration self-adhesive motion situations for neonates

English

Table VI: Disposable sensor reference.

Respironics Sensor Accessory Reference

Sensor Accessory Description	Respironics Model Number	For Use With	Packaging Information
Tape Strips	941	Flex sensors	---
Hydrogel Tape Strips	954	Flex sensors	Packaged with 953, 934, and 935
Reflectance Sensor Holder	952	Reflectance sensor	Packaged with 951
3 foot extension cable	927	All sensors for the 920	---

Table VII: Sensor accessory reference.

B. Finger Clip Sensors

NOTE: For the best results, secure the sensor cable independently from the sensor, preferably around the base of the finger. Make sure that the tape securing the cable does not restrict the blood flow.

NOTE: If the patient's finger is too small or the sensor is not adjusted properly, light may bypass the tissue and result in SpO₂ inaccuracies.

1. Adult Articulated Finger Clip Sensor

Insert finger into the sensor until the end of the finger reaches the finger stop. Ensure that long fingernails are not interfering with proper finger position. The thumb is specifically not recommended for use with the Adult Articulated Finger Clip Sensor.

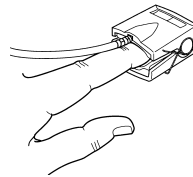


Figure VIII: Adult Articulated Finger Clip Sensor.

2. Pediatric Finger Clip Sensor

Insert finger into the sensor until the end of the finger reaches the finger stop. Ensure that long fingernails are not interfering with proper finger position. The thumb is specifically not recommended for use with the Pediatric Finger Clip Sensor.

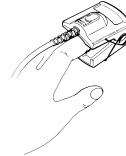


Figure IX: Pediatric Finger Clip Sensor.

C. Flex Sensors

NOTE: For optimum light transmission, attach the sensor on the finger or toe. For best results, secure the cable independently from the sensor. Make sure that the tape securing the cable does not restrict the blood flow.

NOTE: Ensure sensor placement is positioned such that no stray light bypasses the tissue or SpO₂ inaccuracies may result.

1. Adult Flex Sensor

The Adult Flex Sensor is designed for monitoring of adult and pediatric (greater than 20 kg) patients in which patient movement may occur and/or long term monitoring is required. Apply the double stick tape to the side of the sensor which is applied to the patient's finger.

Position the sensor on the top and bottom of the finger or toe as shown in Figure X. Place the light emitter portion on the finger/toe nail side and the detector on the side opposite of the nail. In all sensor placement applications, align the windows (detector and emitter portions of the sensor) over the tissue as illustrated in Figure X. Attach the sensor by wrapping the tape or wrap over the sensor assembly. Wrap the sensor snug, but not so tight as to restrict the blood flow.

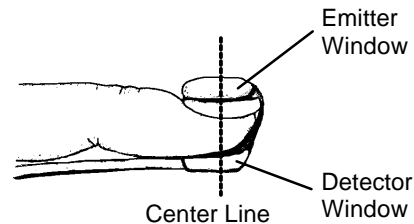


Figure X: Adult Flex Sensor.

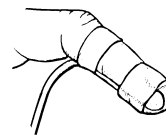


Figure XI: Flex Sensor with tape strain relief.

2. Infant Flex Sensor

NOTE: *The preferred sensor site for infants is on the big toe. This placement provides increased light transmission compared to the foot placement. Refer to Table V for approximate size requirements for infant sensors.*

The Infant Flex Sensor is designed for continuous monitoring of infants where finger tip monitoring is impractical. It is designed for application on the large toe of infants. Apply the Hydrogel Tape Strips to the side of the sensor which is applied to the patient's toe.

Position Infant Flex Sensors on the big toe as illustrated in Figure XII. Make sure that the emitter portion of the sensor is aligned with the detector portion.

Attach the sensor by wrapping the tape or sensor wrap over the sensor assembly snugly, but not so tight as to restrict blood flow. For best results, secure cable independently from the sensor, preferably around patient's ankle or lower leg. Make sure that the tape securing the cable does not restrict the blood flow.



Figure XII: Infant Flex Sensor on big toe.

3. Neonatal Flex Sensor

The Neonatal Flex Sensor is designed for continuous monitoring of neonates where finger tip monitoring is impractical. The Neonatal Flex Sensor is designed for application on the foot of infants. Apply the Hydrogel Tape Strips to the side of the sensor which is applied to the patient's foot.

Position Neonatal Flex Sensors on the foot as far forward near the toes as practical, ensuring sensor light does not pass between the toes as illustrated in Figure XIII. Make sure the emitter portion of the sensor is aligned with the detector portion.

Attach the sensor by wrapping the tape or sensor wrap over the sensor assembly snugly, but not so tight as to restrict blood flow. For best results, secure cable independently from the sensor, preferably around patient's ankle or lower leg. Make sure the tape securing the cable does not restrict blood flow.

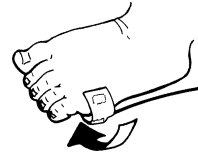


Figure XIII: Neonatal Flex Sensor on foot.

D. Ear Clip and Reflectance Sensors

!	CAUTION	• The Ear Clip and Reflectance sensors are not recommended for pediatric or neonatal use. The accuracy of these sensors has not been established for pediatric or neonatal use.
---	----------------	---

NOTE: *Reflectance and Ear Clip sensors generally do not perform as well as sensors located on the fingers or toes. They are not recommended for applications where the best possible SpO₂ accuracy is important.*

1. Ear Clip Sensor

The Ear Clip Sensor is designed for adults where fingertip monitoring is impractical. Rub the ear lobe vigorously for 5 seconds and then apply the Ear Clip Sensor to the lobe of the ear. Make sure the Ear Clip Sensor is positioned so the LED emitter and detector are completely covered by the earlobe. This ensures no stray light bypasses the earlobe, which can lead to SpO₂ inaccuracies.



Figure XIV: Ear Clip Sensor.

2. Reflectance Sensor

The Reflectance Sensor is for use on well-vascularized skin surfaces. For adults, this will usually be the center of the forehead slightly above and between the eyebrows. The Sensor Holder provides the precise pressure this sensor needs against the skin.

Remove the backing from one side of the double-back tape and apply to the flange of the Sensor Holder. Then remove the back from the other side of the tape. Press the Reflectance Sensor into the foam with the sensor window out, and apply to the patient. Use additional tape to secure the sensor secondarily to the patient to avoid pulling or tipping the sensor.

E. Adult and Pediatric Disposable Sensors



WARNING

- Do not stretch the tape while applying the sensor. This may cause inaccurate readings or skin blisters.

The Flexi-Form[®] Sensors are designed for monitoring adult and pediatric patients as a single patient use sensor and are intended for use where moderate patient movement is expected or cross contamination is a concern.

The preferred application site is the index, middle, or ring finger. However, the thumb or toes may be used where the tissue thickness is between 5 and 21 millimeters. Other sites may not give acceptable results because of inadequate perfusion or inadequate light transmission. The application of these sensors is the same for either the adult or the pediatric patient. The difference is in the size of the sensor. For best results, secure the cable independently from the sensor. Make sure that the tape securing the cable does not restrict the blood flow.


NOTE: *Ensure sensor placement is positioned such that no stray light bypasses the tissue or SpO₂ inaccuracies may result.*




Step 1: Grip the tab on the adhesive backing on the bottom of the sensor and carefully peel it off.



Step 2: Place the patient's finger or toe in the sensor with the nail side up and the tip of the finger or toe centered against the center line marked on the sensor as illustrated. This will ensure vertical alignment between the emitter and detector. Wrap the sensor around the finger. The finger nail should not be covered with tape during this step.

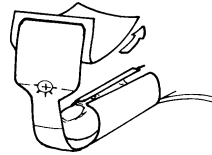

Step 3: Grip the tab on the adhesive backing of the top of the sensor and carefully peel it off.

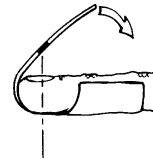

Step 4: Fold the sensor's top over the top of the finger. Ensure that the detector and emitter are vertically aligned as illustrated by the dotted axis line.



Figure XV: Finger Flexi-Form Sensor.

F. Infant and Neonatal Disposable Sensors




WARNING

- Do not stretch the tape while applying the sensor. This may cause inaccurate readings or skin blisters.
- Check application site **frequently** to determine circulation, positioning, and skin sensitivity of the patient. Each patient's sensitivity to Respironics sensors may vary depending on their medical status or the condition of their skin.


1. Infant Flexi-Form Sensor

The preferred application site for the Infant Flexi-Form[®] Sensor is the large toe of infants. Other sites may not give acceptable results because of inadequate perfusion or inadequate light transmission.

For best results, secure cable independently from the sensor, preferably around the patient's ankle or lower leg. Make sure the tape securing the cable does not restrict blood flow.

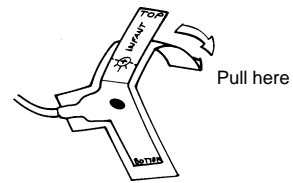

Step 1: Grip the sensor's adhesive backing and peel the adhesive off carefully, from top to bottom, as illustrated.

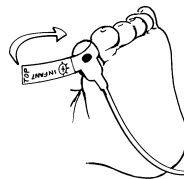

Step 2: Place the sensor on the large toe with the center line indicated on the sensor centered on the side of the large toe, taking care to have the detector centered on the bottom of the toe and the emitter located directly opposite on the toe nail, as illustrated.

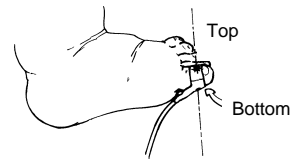



Figure XVI: Infant Flexi-Form Sensor.

Step 3: Wrap the tape around the toe taking care to ensure that the detector and emitter are vertically aligned as illustrated by the dotted axis line.


English

2. Neonatal Flexi-Form Sensor


The preferred application site for the Neonatal Flexi-Form[®] Sensor is the outstep of the foot, close to the toes for neonates. Other sites may not give acceptable results because of inadequate perfusion or inadequate light transmission. For best results, secure the cable independently from the sensor. Make sure that the tape securing the cable does not restrict the blood flow.

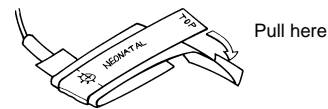

Step 1: Grip the sensor's adhesive backing and peel the adhesive cover off.

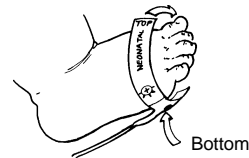

Step 2: Place the sensor on the outstep of the foot just behind the toes (as close to the toes as possible but not over the toes). Align the dashed line mark to the center of the outstep edge of the foot. The light emitter should be on the top and the detector on the bottom.

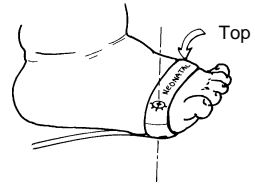

Step 3: Wrap the tape on top and bottom as illustrated.

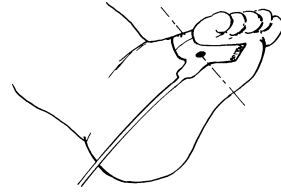



Figure XVII: Neonatal Flexi-Form Sensor.


Step 4: Ensure that the detector and emitter are vertically aligned as illustrated by the dotted axis line.

G. Cleaning the Sensors

 CAUTION	<ul style="list-style-type: none">• Do not immerse the sensors in liquid to clean.• Do not use caustic or abrasive cleaning agents.
--	--

Carefully clean the Respironics reusable sensors with an isopropyl alcohol wipe and ensure that all tape residue is removed. Allow enough time for the sensor to dry thoroughly before reusing. The sensors may be sterilized using ethylene oxide (EtO) (cold cycle).

H. Sensor Compatibility

 WARNING	<ul style="list-style-type: none">• Use only Respironics-manufactured sensors. These sensors are manufactured to meet the calibration requirements for Respironics Pulse Oximeters.
--	---

The 920 is compatible with all Respironics-manufactured sensors (except the Fiber Optic sensors). Refer to Tables V, VI, and VII for a list of the sensors, available sensor accessories, and their model numbers.

VII. Theory of Operation

The pulse oximeter shines light (red and infrared) through the tissue and detects the fluctuating signals caused by arterial blood pressure pulses. Well oxygenated blood is bright red, while poorly oxygenated blood is dark red. The pulse oximeter determines oxygen saturation from this color difference by measuring the ratio of absorbed red and infrared light. Since steady conditions (steady venous blood flow, skin thickness, bone, finger nails, etc.) do not cause fluctuations, they do not affect the saturation reading.

Mathematically:

$$SpO_2 = f \left[\frac{\ln \left(\frac{\text{min}}{\text{max}} \right)_{\text{red}}}{\ln \left(\frac{\text{min}}{\text{max}} \right)_{\text{infrared}}} \right]$$

Note that anything that affects the intensity of the light such as thick or colored skin will affect the maximum and minimum proportionally and thus the ratio max/min does not change. However, if too little light gets through, the pulse oximeter will not work and will signal sensor alarm.



CAUTION

- The Respirationics 920 Pulse Oximeter is calibrated to determine the percentage of arterial oxygen saturation of functional hemoglobin. Significant levels of dysfunctional hemoglobin such as carboxyhemoglobin or methemoglobin may affect the accuracy of the measurement.
- Cardiogreen and other intravascular dyes, depending on the concentration, may affect the accuracy of the SpO₂ measurement.

Pulse oximeters use two different wavelengths of light (colors) and thus have the ability to determine one component of blood. The Respirationics 920 Hand Held Pulse Oximeter is calibrated to closely approximate functional oxygen saturation values. The 920 oxygen saturation values will closely approximate laboratory instrument fractional saturation values if the dysfunctional hemoglobin saturation levels are negligible. If the dysfunctional hemoglobin is carboxyhemoglobin or methemoglobin, then the difference between the oxygen saturation value displayed by the 920 and the oxygen saturation values determined by the laboratory instrument will be greater as the dysfunctional hemoglobin levels rise approximately in accordance with the following formulas:

$$SpO_2 = O_2Hb + COHb + MetHb$$

$$SaO_2 = 100 \times O_2Hb / (100 - COHb - MetHb)$$

Where:

- SpO₂ = Respirationics 920 determined and displayed oxygen saturation in percent
- O₂Hb = Fractional oxyhemoglobin saturation in percent
- COHb = Carboxyhemoglobin saturation in percent
- MetHb = Methemoglobin saturation in percent
- SaO₂ = Functional oxygen saturation in percent

Example 1

- O₂Hb = 96
- COHb = 0.5
- MetHb = 0.6
- SpO₂ = 97
- SaO₂ = 97.07

Example 2

- O₂Hb = 88
- COHb = 8
- MetHb = 2
- SpO₂ = 98
- SaO₂ = 97.78

The mathematics are fixed in the pulse oximeter hardware and software. Thus no field calibrations are needed or are they possible. There are no adjustable parts within the pulse oximeter that affect the calibration.

The function, *f*, does depend on the color of the LEDs producing the light. These colors are fixed by the manufacturing processes and material. The sensors are checked for correct operation before shipping so no adjustment or calibration is needed or possible.

In summary, because the pulse oximeter does all critical computations in software and there are no critical parts to drift; no re-calibration is needed. Also, since the pulse oximeter reading does not depend on the absolute light intensity, but only on the fluctuations, then extremely thick skin, finger nails, colored skin, etc., do not affect the readings.

VIII. Specifications

1. Oxygen Saturation Range (SpO₂)	0 to 100%
2. Pulse Rate Range	18 to 300 Pulses Per Minute
3. Displays	
Patient Indicator	Perfusion LED
Digital Displays	3-digit 7-segment LEDs
4. Measurement Wavelengths	
Red	660 nanometers
Infrared	910 nanometers
5. Accuracy	
SpO₂	
(± 1 Standard Deviation)**	70 to 100% ± 2 digits for adults using Finger Clip Sensors
	70 to 100% ± 3 digits for adults using Flex, Flexi-Form, or Reflectance Sensors
	70 to 100% ± 4 digits using Ear Clip Sensors
	70 to 95% ± 3 digits for neonates using infant or neonatal sensors
	Below 70% is not specified for all sensors
Pulse Rate	± 3% ± 1 digit

** Standard Deviation is a statistical measure: up to 32% of the readings may fall outside these limits.

6. Temperature	
Operating	-20 to +50 °C
Non-operating	-30 to +50 °C
7. Humidity	
Operating	10 to 90% non-condensing
Non-operating	10 to 95% non-condensing
8. Power Requirements	6 AA alkaline batteries 100 hours typical operation (maximum display brightness) 200 hours typical operation (minimum display brightness)
9. Patient Isolation	> 12 MΩ
10. Leakage Current	Not applicable
11. Dimensions	3" wide x 6" high x 1" deep 8 cm x 15 cm x 2 cm
12. Weight	10 oz. (280 g) with batteries



IX. Service



CAUTION

- 920 Hand Held Pulse Oximeters are sensitive and must be repaired by knowledgeable and specially trained personnel only. Any sign or evidence of opening the system, field service by non-Respironics personnel, tampering, or any kind of misuse or abuse of the system, shall void the warranty in its entirety.

The advanced digital circuitry within the 920 requires no periodic maintenance or calibration other than battery replacement.

Respironics does not recommend field repair of the 920 Hand Held Pulse Oximeter. The circuit board in the 920 series is a multi-layer board using traces 0.01" wide. Due to the very small trace size, extreme care must be used when replacing components to prevent permanent non-repairable damage to the circuit board. Most components are surface-mounted and require special hot air jet soldering and desoldering equipment. After any repairs are made, the pulse oximeter must be tested to ensure correct operation.

NOTE: *All repair work on the 920 should be done by trained Respironics personnel. For Respironics Technical Support contact:*

Technical Support Telephone: 1-800-421-8754
(USA and Canada)

Technical Support Telephone: 1-770-499-1212
(International)

Customer Service Telephone: 1-800-345-6443
(USA and Canada)

Customer Service Fax: 1-800-886-0245
(USA and Canada)

www.respironics.com

All non-warranty work shall be done according to Respironics standard rates and charges in effect at the time of delivery to Respironics. All repairs include an 18-hour burn-in and a complete retest of the pulse oximeter using factory test fixtures.

X. Warranty

RESPIRONICS, INCORPORATED, (Respironics) warrants to the purchaser, for a period of three years from the date of delivery, each system exclusive of sensors, cables, and batteries. Respironics shall repair all systems found to be defective in accordance with this warranty, free of charge, for which Respironics has been notified by the purchaser by serial number that there is a defect, provided said notification occurs within the applicable warranty period. This warranty shall be the sole and exclusive remedy by the purchaser hereunder for any systems or accessories delivered to the purchaser which are found to be defective in any manner whether such remedies be in contract, tort or by law.

This warranty excludes cost of delivery to and from Respironics. All repaired units shall be received by the purchaser at Respironics 's place of business. For any system or accessory sent to Respironics for warranty repair which is found to be within specification, the purchaser agrees to pay \$100.00 (USD).

These systems are sensitive and must be repaired by knowledgeable and specially trained personnel only. Accordingly, any sign or evidence of opening the system, field service by non-Respironics personnel, tampering, or any kind of misuse or abuse of the system, shall void the warranty in its entirety.

All non-warranty work shall be done according to Respironics standard rates and charges in effect at the time of delivery to Respironics.

DISCLAIMER/EXCLUSIVITY OF WARRANTY

THE EXPRESS WARRANTIES SET FORTH IN THIS MANUAL (SECTION X) ARE EXCLUSIVE AND NO OTHER WARRANTIES OF ANY KIND, WHETHER STATUTORY, WRITTEN, ORAL, OR IMPLIED INCLUDING WARRANTIES OF FITNESS FOR A PARTICULAR PURPOSE OR MERCHANTABILITY SHALL APPLY.

XI. Accessories

The following Respironics accessories function with the Model 920 Hand Held Pulse Oximeter:

<u>Model</u>	<u>Pulse Oximeters and Printer</u>
920	One 920 Hand Held Pulse Oximeter; Adult Articulated Finger Clip Sensor (Model 936); and Instruction and Service Manual (Model 924); Six AA Alkaline Batteries
920M	One 920M Hand Held Pulse Oximeter with 18-Hour Memory; Adult Articulated Finger Clip Sensor (Model 936); and Instruction and Service Manual (Model 924); Six AA Alkaline Batteries
920P	Spot-Check Printer for either the 920 or 920M

<u>Model</u>	<u>Reusable Sensors</u>
936	Adult Articulated Finger Clip Sensor
936P	Pediatric Finger Clip Sensor
953	Adult Flex Sensor, 38mm/ (finger/toe) straight cable with 954 Hydrogel Tape Strips (25)
934	Infant Flex Sensor, 30mm/ (toe), right angle cable, with 954 Hydrogel Tape Strips (25)
935	Neonatal Flex Sensor, 38mm/ (outstep), right angle cable, with 954 Hydrogel Tape Strips (25)
948	Ear Clip Sensor
951	Reflectance Sensor, adult, includes 10 holders (Model 952)

<u>Model</u>	<u>Disposable Sensors</u>
953D	Adult Finger Flexi-Form [®] Sensor ¹ , 10 per box
937D	Pediatric Finger Flexi-Form [®] Sensor, 10 per box
934D	Infant Toe Flexi-Form [®] Sensor, 10 per box
935D	Neonatal Foot Flexi-Form [®] Sensor, 10 per box
9000D	Flexi-Form [®] Sensor Assortment Pack, 10 per box

<u>Model</u>	<u>Related Accessories</u>
954	Hydrogel Tape Strips, 25 per bag
941	Tape Strips, double-stick, 100 per box
952	Reflectance Sensor Holders (10) for the 951
927	Patient Cable (3 foot extension)
956	Carrying Case for the pulse oximeter, printer, extra paper, and batteries
957	Thermal Paper for the 920P (box of 20)
924	Instruction and Service Manual (Pulse Oximeter)
958	Instruction and Service Manual (Printer)

<u>Model</u>	<u>Profox Software and Accessories</u>
920M-40	Profox Software for the 920M, includes Download Cable (Model 920M-31)
920M-31	Profox Software Download Cable
920M-42	Profox Instruction Manual

<u>Model</u>	<u>Test Accessories</u>
960	Finger Phantom® Pulse Oximeter Testing System ¹ (Starter Kit) (Includes Models 960R, 960H, and 961, and a durable carrying case)
960R	Finger Phantom Replacement Sets (Three Finger Phantom Replacements: 1 of each @ 97%, 90%, and 80% SpO ₂)
960H	Finger Phantom Holder
961	Finger Phantom Instruction and Service Manual
955	Patient Simulator



¹ *Flexi-Form*® and *Finger Phantom*® are registered trademarks of Nonin Medical, Inc.

For further information about Respironics parts and accessories contact your local distributor.

If you do not have your local distributor information, call Respironics and ask for your local distributor's name and phone number.

XII. Troubleshooting Guide

Symptom	Possible Cause	Possible Solution
920 will not power up	Batteries are completely depleted	Replace all six batteries of the 920
	Incorrect battery installation	Verify battery sequence. Refer to Figure I: Replacing Batteries in the 920
	Battery door is missing metal contact	Replace the battery door
Numeric displays are blinking at once per second	Battery voltage is low	Replace all six batteries of the 920
	Incorrect battery installation	Verify battery sequence. Refer to Figure I: Replacing Batteries in the 920
Minus sign appears in SpO ₂ display	Sensor fault exists. Sensor may have become dislodged from 920 or from the patient	Verify sensor is connected to the 920 and the patient correctly; Try a new sensor if condition persists
Segments of SpO ₂ or © display are missing	Defective LED displays	Displayed values may not be reliable; discontinue use of 920
Displayed pulse rate does not correlate to pulse rate displayed on ECG monitor	Excessive motion at sensor site may be prohibiting the 920 from acquiring a consistent pulse signal	Eliminate or reduce cause of motion artifact <u>or</u> reposition sensor to new sensor site where motion is not present
	Patient may have an arrhythmia resulting in some heart beats that do not yield a perfusion signal at sensor sight	Examine the patient: Condition may persist even though both monitors are functioning properly if patient's arrhythmia persists
	Non- Respirationics sensor is being used	Replace sensor with a Respirationics sensor
	ECG monitor may not be functioning properly	Examine the patient: Replace ECG monitor <u>or</u> refer to operator's manual for ECG monitor

Troubleshooting Guide (continued)

Symptom	Possible Cause	Possible Solution
Erratic © display and/or yellow perfusion LED during concurrent use of electro-surgical equipment (ESU)	ESU may be interfering with oximeter performance	Examine the patient: move oximeter, cables, and sensor as far away from ESU as possible <u>or</u> refer to the ESU operator's manual
Perfusion is blinking yellow with each pulse	Perfusion signal at sensor site is marginal	Examine the patient: reposition sensor <u>or</u> select alternate sensor site
Unable to obtain green perfusion	Low patient pulse strength	Reposition sensor on patient
	Sensor site poorly perfused	
	Sensor not correctly positioned	
	Sensor attached too tightly or tape or other items are restricting perfusion at sensor site	Reapply sensor, select alternate sensor site, or remove restrictive material from sensor site
	Circulation reduced due to excess pressure between the sensor and a hard surface	Allow sensor and finger to rest comfortably on surface
	Excessive ambient light	Reduce ambient light
	Excessive patient motion	Reduce patient motion
	Sensor applied to a polished fingernail	Remove fingernail polish
Interference from: <ul style="list-style-type: none"> • arterial catheter • blood pressure cuff • electro-surgical procedure • infusion line 	Reduce or eliminate interference	

English

Troubleshooting Guide (continued)

Symptom	Possible Cause	Possible Solution
Perfusion LED is blinking red and SpO ₂ and ♥ displays show dashes	Inadequate perfusion signal at sensor site	Examine the patient: reposition sensor <u>or</u> select alternate sensor site
	Excessive motion at sensor site may be prohibiting 920 from acquiring a consistent pulse signal	Eliminate or reduce cause of motion artifact or reposition sensor to sensor site where motion is not present
Printer not printing out after changing <i>P r n</i> mode	Printer mode did not get updated internally	Reset pulse oximeter by turning it off and then on

If any of these solutions do not correct the problem with your 920, please contact Respiration Technical Support.